



FlexiOH[®]

Light Cured Polymer Immobilizer

SHORT ARM IMMOBILIZER

INSTRUCTIONS FOR USE

Please read these instructions carefully before application. Stickers should be used from the label to keep record of product identity.

FlexiOH[®]

Light Cured Polymer Immobilizer

GENERAL DESCRIPTION

FlexiOH is the first of its kind - an orthopedic immobilizer, that is breathable and washable. The design of FlexiOH is rigid enough to hold the fractured part as well as ensures skin ventilation. FlexiOH comes in a flexible form which is wearable. Light treatment completes the curing process and makes the immobilizer rigid. Curing is done by Rizycure , which is a handheld high intensity portable visible blue light system. FlexiOH is made of silicone rubber, light curable resin and nitrile rubber foam. The device is for single use only and supplied non-sterile. The curing process is quick and takes around 5-10 minutes for curing depending on the size. Post treatment the FlexiOH can be removed easily without the use of any other cast cutter or spreader.

The device is tested for biocompatibility and radiolucency.

KEY FEATURES



Washable



Breathable



Easy Application



Comfortable



Safe



Latex Free

SHORT ARM IMMOBILIZER

FlexiOH Short Arm Immobilizer comes in 9 different sizes: 2XS, XS, S, M, L, XL, SM, SL and SXL.

INTENDED USE

The FlexiOH – light cured polymer immobilizers are intended to be used for immobilization of the body parts having musculoskeletal injuries like fractures, sprains, ligament injuries as well as for pre-surgical and post-surgical immobilization.

INDICATION FOR USE

The FlexiOH Short Arm Immobilizers are intended to be used for immobilization of wrist joints and closed fractures of radius/ulna. Short Arm Immobilizers are also used as a removable splint during rehabilitation. The immobilizers can be used on patients suffering from:

- Non-displaced / stable / minimally displaced / buckle fracture of distal radius.
- Carpal bone fracture other than scaphoid / trapezium.

CONTRAINDICATIONS

The FlexiOH - light cured polymer immobilizers should not be used where the device comes in contact with breached skin.

PATIENT POPULATION

Patients suffering from upper limb bone fractures - wrist joints and closed fractures at radius/ulna.


INTENDED USERS

A health care professional (HCP) such as an orthopedic surgeon, emergency room doctor, physician assistant, orthopedic technician or nurse practitioner.

SIZE CHART (IN CM)

	Mid Palm Circum.	Forearm Length
2XS	9-11	14-18
XS	11-14	18-21
S	14-17	23-26
M	17-18.5	26-29
L	18.5-22.5	29-30
XL	22.5-24.5	30-33
SM	17-18.5	23-27
SL	18.5-22.5	27-29
SXL	22.5-24.5	29-30

MEASUREMENT GUIDE

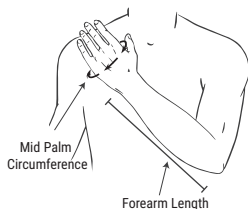

Notes :


* Above sizes fit better as per dimensions mentioned in the table.

* We advise healthcare professionals to choose the size as per patient's comfort.

APPLICATION PROCESS


Follow the steps to apply FlexiOH Short Arm Immobilizer :

(a)

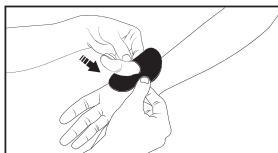


Place the thumb cushion pad over the thumb.

(b)

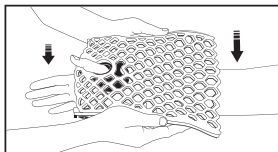


Place FlexiOH Short Arm immobilizer on the hand such that thumb holds it like an anchor.

(c)

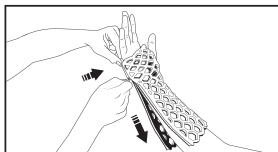


Start zipping just above the wrist area, follow the ulnar line and apply the zip carefully.

(d)

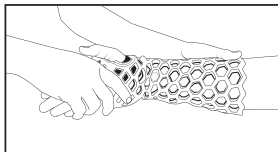

After completing the zipping process, adjust the immobilizer as per required position.

(e)

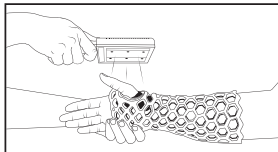

Start curing around the palm area first using RizyCure. Then cure wrist and forearm area for about 5-10 minutes until it gets rigid.

NOTE : While curing maintain the distance between FlexiOH and RizyCure - less than 1 cm for rechargeable RizyCure and 10 cms for corded Rizycure.

REMOVAL PROCESS

- Step 1 : Cut the fabric liner of the zip using scissors.
- Step 2 : Break the rib of FlexiOH Immobilizer on the opposite side of zipper line using bone cutter pliers (Bone Cutting Forceps Straight Double Action).
- Step 3 : Apply gentle force to open the immobilizer from zip side to remove it.
- Step 4 : Dispose the used immobilizer as per local biomedical waste norms.

WARNINGS AND PRECAUTIONS

- For best fit, choose size before application of FlexiOH immobilizer from the size chart. Make sure that the patient feels comfortable before the curing process.
- Inspect FlexiOH for any manufacturing defect i.e. foam damage, leakage, etc. Do not use the product for treatment if found with such defect. Dispose such defective product only after curing as per local biomedical waste norms.
- In case, the box is damaged but the product is intact in the sealed sleeve bag, product can be used for the treatment.
- Ensure that the zipper is always aligned to the medial side when in supine position.
- RzyCure light device is used to cure FlexiOH immobilizer. Usually, it takes 5-10 minutes for curing, it might take shorter or longer time duration depending on the size of the immobilizer.
- The curing process is quick, hence always ensure that the position of hand and fingers are properly aligned before you start the curing process.
- After curing keep the patient's hand and fingers static until 10 minutes to allow complete curing of the cast.
- Do not reuse, as it may lead to cross-contamination or infection.
- Do not use the device on patient allergic to the materials used in the device.

CAUTION : In any case of constant discomfort, skin rashes, increased pain, numbness or redness of the skin, patients should consult immediately to HCPs. FlexiOH should be removed immediately if required as per HCP's opinion.

ADVERSE EFFECTS:

- The known adverse events related to cast application are - Itching, Pain, Rash, Odor, Skin breakdown, Joint stiffness, Compartment syndrome, Muscle atrophy, Distal oedema, Development of swelling.
- **Skin Complication:** Erythema, Pressure sore, Dermatitis, Skin allergic reaction.

STORAGE CONDITIONS

Keep the device away from sunlight until final application and curing process.

MR COMPATIBILITY

All the materials used in the FlexiOH are non-ferrous and MR compatible.

REPORTING OF ADVERSE EVENTS

In case of any event please notify to 'care@orthoheal.com' or to the respective country regulatory authority.

INSTRUCTIONS FOR PATIENT

Do's

- Normal day-to-day activities which require less movement of the fractured part can be performed while wearing immobilizer.
- Bathing and swimming in shallow pool is allowed.
- Dry immobilizer with hair dryer if required.
- Clean immobilizer regularly to maintain skin hygiene.
- Keep injured limb in elevated position in order to reduce swelling.

Dont's

- Don't touch upper surface of immobilizer during light curing process. Higher temperature may cause burns.
- Don't lift any weighted object with the fractured limb without your doctors permission.
- Don't try to remove immobilizer without doctor's permission.
- Avoid any strenuous musculo-skeletal activities.
- Don't remove / peel off cushioning foam from immobilizer, it may expose the hard edge or surface to the skin.
- Don't put unnecessary pressure on the immobilizer.
- Don't go near fire, as FlexiOH may catch fire.

Contact health care professional (HCP) in case the immobilizer is too loose/ too tight/ broken / crack and increased swelling or redness of skin is observed.

SYMBOL GLOSSARY



Medical Device



Caution: Federal law restricts this device to sale by or on the order of a HCP



Do not re-use.



Do not use if package is damaged and consult instructions for use



Non-sterile



Consult instructions for use.



Device is not made with natural rubber latex.



Keep away from sunlight.



Manufacturer



Date of Manufacture



Use-by Date



Catalogue Number / Reference code



Batch code / Lot Number



Serial Number



Unique device identifier



Authorised representative in the European Union

Copy of the design or technology may infringe IP rights.



FDA
L I S T E D



MANUFACTURER

JC OrthoHeal Private Limited
P7 - P8/A, Biotech Park Phase - I,
GIDC Savli, Manjusar,
Vadodara - 391 775. Gujarat, India.

Call: +91 85305 30885
Email: india@orthoheal.com

FOR COMPLAINT AND FEEDBACK

Call: +91 911 811 311 0
Email: care@orthoheal.com



AF Pharma Service Europe SL
Muntaner 281,
Barcelona, 08021,
España

