 <b>JC OrthoHeal Private Limited</b>	<b><i>Work Instruction</i></b>	Proc. No.: W/QAC/05
	<b>Title:</b> Work Instruction for RizyCure Cleaning	Rev No.: 00
		Date: 18.08.2023

## 1. Objective

To prevent transmission of infection, the RizyCure is thoroughly cleaned periodically as per the hospital/clinic device maintenance protocol.

## 2. Scope

This cleaning work instruction shall be applicable to the RizyCure light ,adapter and connecting wire.

- Goggles should be cleaned after every use and disinfected periodically


## 3. Responsibility

Individuals responsible for device maintenance and hygiene protocol.

## 4. Process

### 4.1. Preparation

- 4.1.1. Turn off the power and plug off adapter from the main power supply(if plugged in).
- 4.1.2. Wait until the RizyCure device and adapter cools down.
- 4.1.3. Perform hand hygiene and put on gloves.
- 4.1.4. Put RizyCure and its wire adapter on a clean surface.
- 4.1.5. Use disinfectant wipes as prescribed below for cleaning of the RizyCure light, adapter and connecting wire.
- 4.1.6. Use a gentle rub on RizyCure light and adapter by disinfectant wipe.
- 4.1.7. Use additional wipe if required for clean RizyCure light ,adapter and connecting wire.
- 4.1.8. Make sure that RizyCure adapter and connecting wire are properly wet (please note: disinfectant liquid should not be poured directly on the surface or inserted inside the RizyCure and adapter).
- 4.1.9. Allow the disinfectant to dry and wait as per disinfectant manufacturer guidelines.
- 4.1.10. Now put RizyCure light, adapter and connecting wire in RizyCure case for future use.
- 4.1.11. Discard wipes, remove gloves and perform hand hygiene.

 <b>JC OrthoHeal Private Limited</b>	<b><i>Work Instruction</i></b>		Proc. No.: W/QAC/05
	<b>Title:</b> Work Instruction for RizyCure Cleaning		Rev No.: 00
			Date: 18.08.2023

#### 4.2. Recommended Disinfectant products

##### 4.2.1. Use below wipes for disinfection.

Manufacturer	Name	Description	Contact Time
PDI Healthcare	Super Sani-Cloth Germicidal Wipes	SKU # A22480	3-4 Minutes

PDI Healthcare	Easy Screen® Cleaning Wipe	SKU # P03672	2-3 Minutes
----------------	-------------------------------	--------------	-------------



Do not use harsh cleaners, solvents, or detergents. Failure to do so could result in equipment damage.



The front lens is supplied with a protective hard coat to resist scratching. Never use abrasive cleaners on the front lens. Failure to do so could result in equipment damage.



Do not expose the unit to excessive moisture. Failure to do so could result in personal injury and/or property damage